

# Access Lift - Quick Guide Manual




## Standard Operating Procedure

<p><b>1</b> Lift Out</p>	<p><b>4</b> Pull Out Extension</p> <p>Ensure Aluminium Drops and Locks</p>	<p><b>2</b> Power Up to reach</p>	<p><b>5</b> Power Down</p>
<p><b>3</b> Raise Up Handrails</p> <p>Ensure Handrails are Correctly Engaged</p>	<p><b>6</b> Power Up</p>	<p><b>7</b> Lift Up Aluminium Push Extension In</p>	<p><b>8</b> Fold Down Handrails</p>
<p><b>9</b> Lift In to Stow</p> <p>Check Click!</p>		<p><b>9</b> Lift In to Stow</p> <p>Check Click!</p>	


## Manual Override Procedure

<p><b>Lift Out</b></p> <p><b>1</b> Lift</p> <p><b>2</b> Pull</p>	<p><b>Lift Down</b></p> <p><b>Lift Up</b></p> <p>Turn 1/4 "max" anti-clockwise</p> <p>1/4 "max"</p>	<p><b>Handrails Up/ Extension Out</b></p> <p>1</p> <p>2</p> <p>Waist Height</p>	<p><b>Extension In/ Handrails Down</b></p> <p>2</p> <p>1</p> <p>Waist Height</p>
<p><b>Check-Down</b></p> <p>Click!</p>		<p><b>Push Lift In</b></p>	


## INTENDED USE

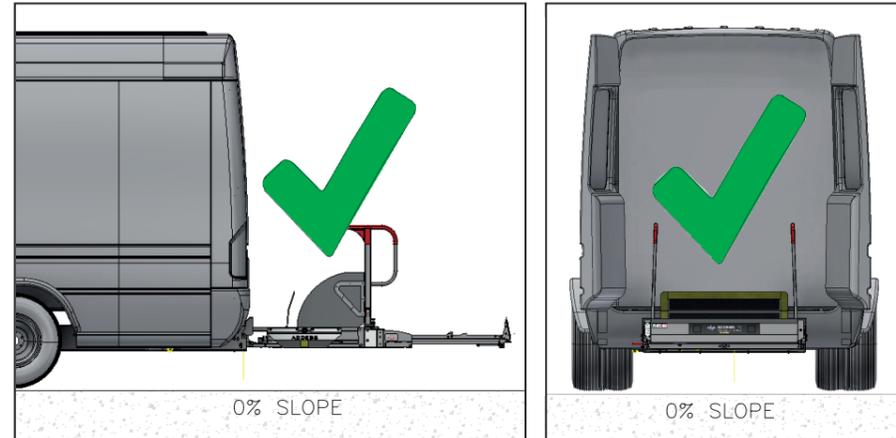


## IMPROPER USE

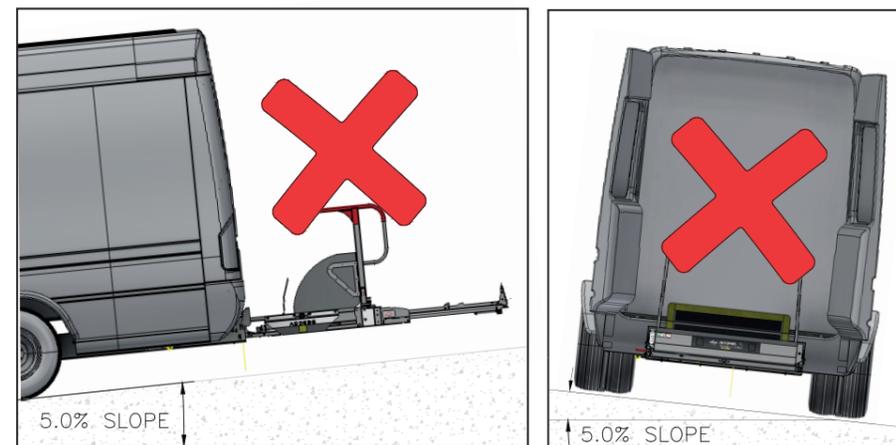


## SAFETY DEVICES

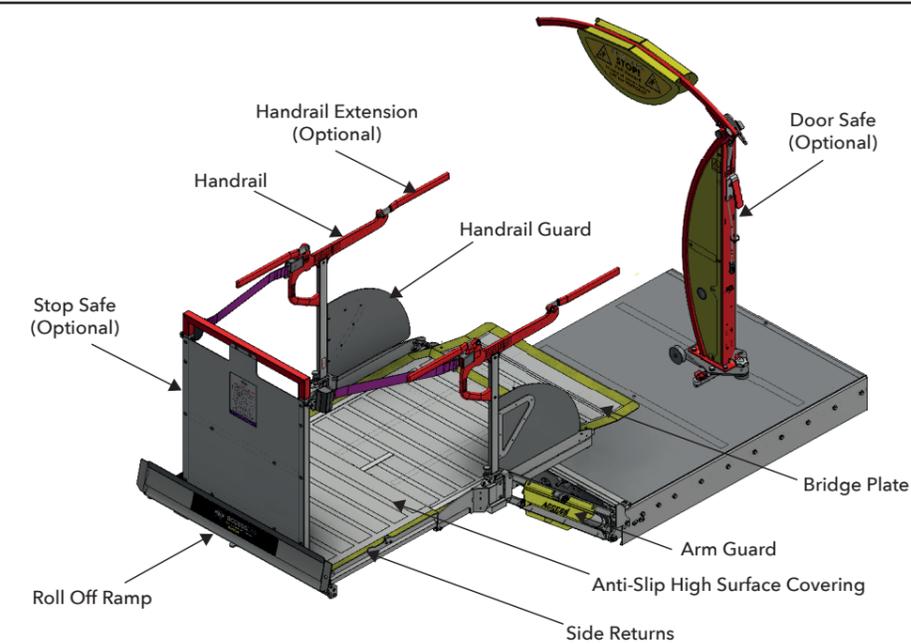


## PASSENGER LOADING OPTION 1

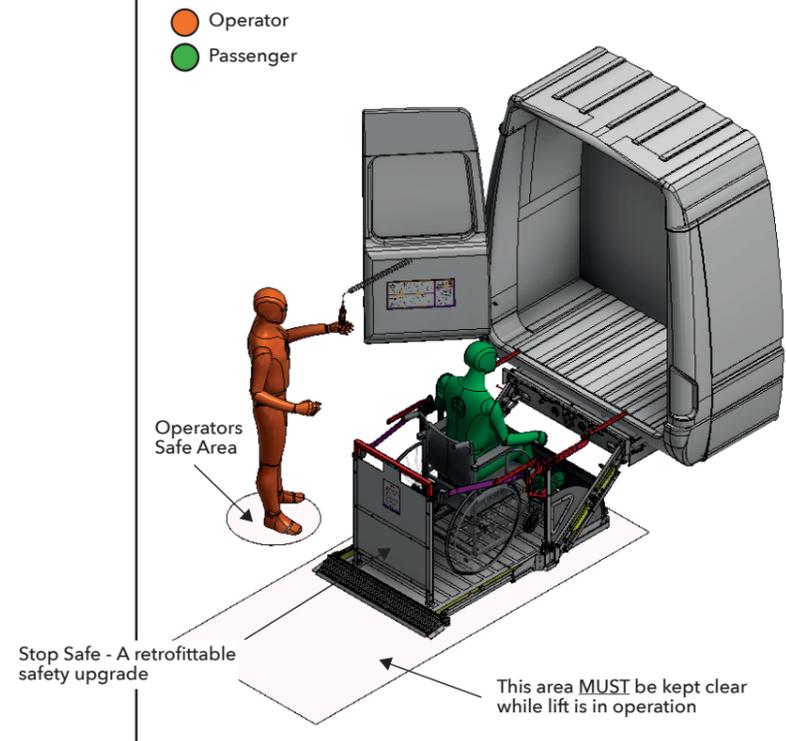


## HANDSET CLIP LOCATION

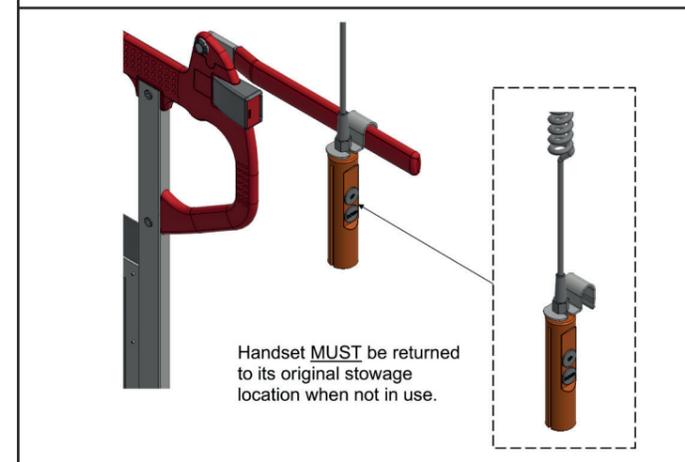

## LIFT ISOLATOR

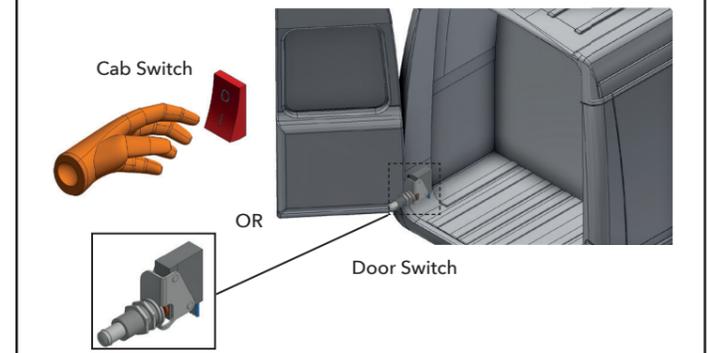

## PASSENGER LOADING OPTION 2

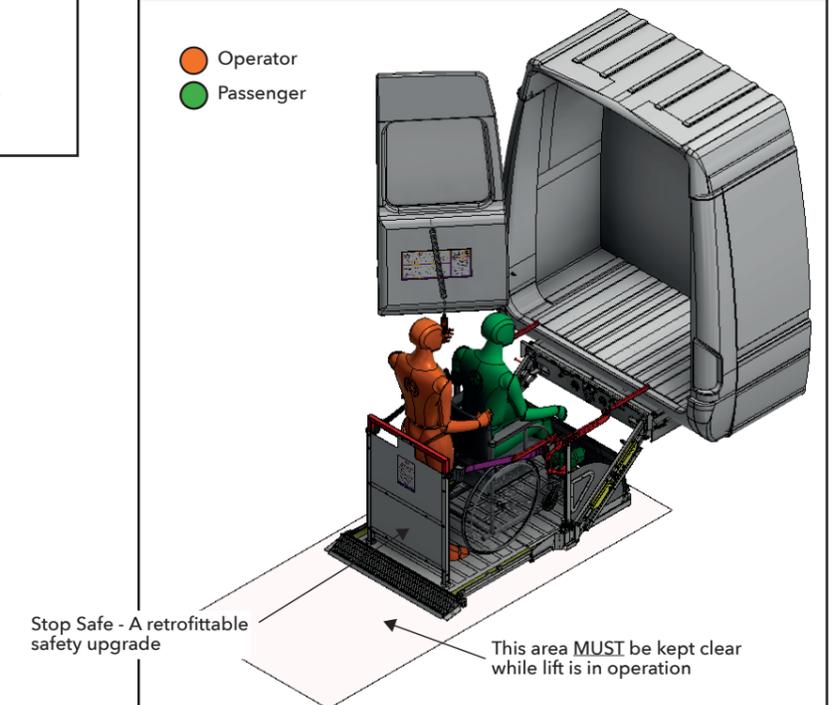

- The lift must be operated by fully trained and competent operators.
- Suitable lighting MUST be provided to enable the operator to use the lift safely.
- The emission sound when the lift is in operation should not exceed 80dB.
- It is the operators responsibility to ensure that all safety features fitted are used appropriately and that all safety warning devices, safety labels / information is/are present and correctly displayed at all times.
- The platform lift is a mechanical device controlled by an advanced hydraulic / electrical system that enables passengers to be elevated safely from the ground into the vehicle.